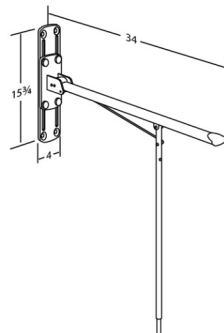




PRESSALIT®

Product Item number	VALUE fold down grab bar, height adjustable R1111000
Color Product description	<p> VVS:615722410 NRF:6191644 RSK:7972413 EAN:5708590176437 HMI: 25414 NBS:N13:436 FDA:ILS/D135866 White </p>  <p> Fold down grab bar with leg, height adjustable 6". Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately. </p> <p> Fold down grab bar with leg, height adjustable 6". Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately. </p>
Adjustability Adjustment height Product capacity	<p>Height adjustable</p> <p>5,91 inch</p> <p> Maximum load: 300 lbs. When the fold down grab bar is loaded with 300 lbs, each mounting bolt is loaded with 69 lbs. </p>
Materials Ergonomy	<p>Basic material: Steel</p> <p>Surface material: Rilsan</p> <p>End cap: Nylon (PA)</p> <p> Provides firm support when you are sitting, standing, or transferring to and from a toilet. </p> <p> Good, broad support surface for the hand and arm. The rounded form gives a firm and comfortable grip. </p>



PRESSALIT®

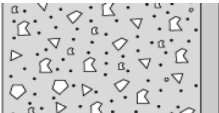
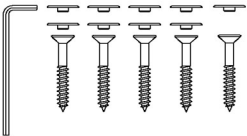

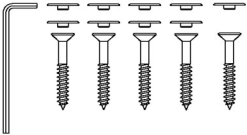
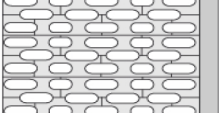
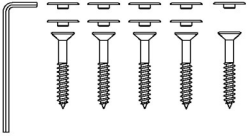
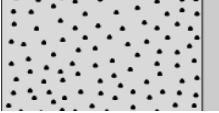
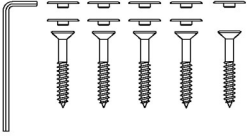
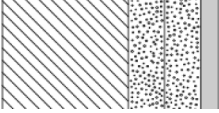
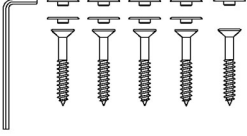
Product	VALUE fold down grab bar, height adjustable
Item number	R1111000
Features	<p>The grab bar can fold up when not in use or to make more space.</p> <p>With leg for extra support.</p> <p>Height adjustable 6".</p>
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: CaviCide</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
Declaration of conformity	This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.



PRESSALIT®

Product	VALUE fold down grab bar, height adjustable
Item number	R1111000

Mounting in different wall types

	<p>Concrete >B25</p> <p>V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p>Solid Brick >= Mz12</p> <p>V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p>Solid Sand Lime Brick >= Hlz12</p> <p>V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p>Aerated Concrete P4</p> <p>V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</p> <p>V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	



PRESSALIT®

Product Item number	VALUE fold down grab bar, height adjustable R1111000
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</p> <p>Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</p> <p>Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>
	<p>Drywall (2 x 12,5 mm) water-resistant</p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S
 Pressalitvej 1
 DK-8680 Ry
 Tel.: +45 8788 8788
 Fax: +45 8788 8789
 E-mail: pressalit@pressalit.com

PRESSALIT®