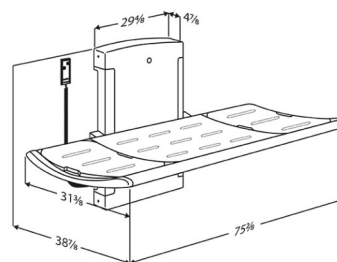




# PRESSALIT®

<b>Product</b> <b>Item number</b>	Adult changing table 3000, electrically height adjustable R8520021
<b>Color</b>	EAN:5708590265629 FDA:INQ/D124319  Lead grey
<b>Product description</b>	<p><b>Discontinued per 31 Dec 2018 and replaced by R8539</b>          Adult changing table 3000, powered, height adjustable 27.6"; from 11.8" - 39.4", with wired hand control          Foldable. Incl. water collection tray with integrated outlet.          Surface length 70.9", outer measurements 38.9" x 75.2".          Excl. mounting kit.          See Technical specifications (pdf) for information on mounting kits suitable for various wall types.</p> <p><b>Discontinued per 31 Dec 2018 and replaced by R8539</b>          Adult changing table 3000, powered, height adjustable 27.6"; from 11.8" - 39.4", with wired hand control          Foldable. Incl. water collection tray with integrated outlet.          Surface length 70.9", outer measurements 38.9" x 75.2".          Excl. mounting kit.          See Technical specifications (pdf) for information on mounting kits suitable for various wall types.</p>
<b>Adjustability</b> <b>Adjustment height</b>  <b>Product capacity</b>  <b>Speed up</b>  <b>Noise level</b>	Height adjustable  27,58 inch  Maximum load: 440 lbs. When the table is loaded with 440 lbs, each mounting bolt supports 57 lbs. <i>This product has been tested statically with a load of 1100 lbs as the IEC 60601-standard requires testing with a safety factor.</i> Speed up: 0,59 inch/s Speed down: 0,79 inch/s  48 dB
<b>Materials</b>	Aluminium, polyurethane foam, plastics, powder-coated steel





<b>Product</b>	Adult changing table 3000, electrically height adjustable
<b>Item number</b>	R8520021
<b>Ergonomy</b>	<p>Pressalit offers a range of high quality, fixed or power adjusted changing tables designed to provide a stable platform suitable for changing adults and children in a variety of settings: schools, day care centers, institutions, hospitals and private homes.</p> <p>The changing table 3000 series provides maximum safety and comfort. Besides an award winning design, the series includes comfortable and durable surfaces, including polyurethane foam and aluminum.</p> <p>The flexibility of the changing table provides optimal working conditions for the caregiver as well as a high level of comfort and safety for the user.</p>
<b>Features</b>	<p>A water collection tray with integrated water outlet prevents water from splashing on caregiver or the floor. The flexible hose attached to the center mounted outlet of the water collection tray, can be connected to either an outlet in the floor or on the wall. Adjustable neck support which can be moved to either end of the 70.9" changing tables.</p> <p>The slightly curved form makes the table extremely comfortable and safe.</p> <p>The integrated pneumatic cylinders enable the table to be folded out of the way when not in use to maximize floor space.</p>
<b>Mains connection (V)</b>	120
<b>Actuator (motor)</b>	LA31 Careline, (Linak), 6000 N, IPX6
<b>Control box</b>	CB9 Careline, (Linak), IPX6
<b>Wired remote control</b>	HB3 (Linak), IP66
<b>Power consumption (maximum load)</b>	Power consumption by maximum load: 70 W Power consumption by standby: 2 W
<b>Warranty</b>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.</p>



# PRESSALIT®

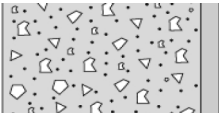
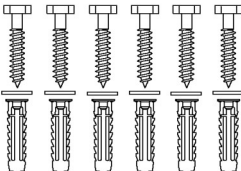

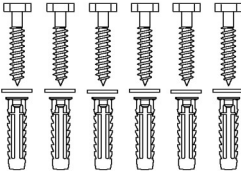
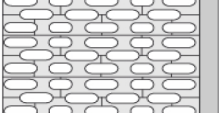
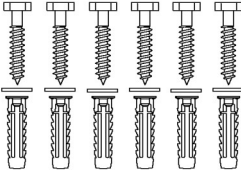
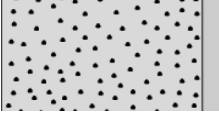
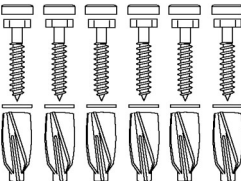
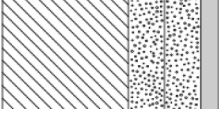
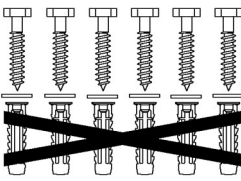
<b>Product</b>	Adult changing table 3000, electrically height adjustable
<b>Item number</b>	R8520021
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>



PRESSALIT®

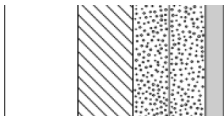
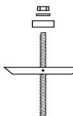

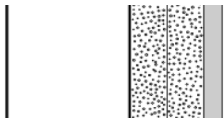
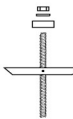

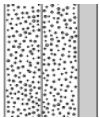

<b>Product</b>	Adult changing table 3000, electrically height adjustable
<b>Item number</b>	R8520021

## Mounting in different wall types

	<p><b>Concrete &gt;B25</b></p> <p>V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.</p>	
	<p><b>Solid Brick &gt;= Mz12</b></p> <p>V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.</p>	
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.</p>	
	<p><b>Aerated Concrete P4</b></p> <p>V8658 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs GB14. To be ordered separately.</p>	
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. NB! Plugs are not needed. To be ordered separately.</p>	



# PRESSALIT®

<b>Product</b> <b>Item number</b>	Adult changing table 3000, electrically height adjustable R8520021		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b> Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.		

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

[pressalit.com](http://pressalit.com)