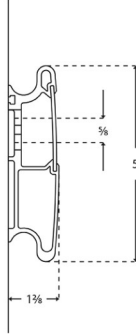




Product Item number	PLUS wall track, 94.5" R9824112
Color	<p>VVS:615841124 NRF:7501019 EAN:5708590305851 HMI: 71745</p> <p>Anthracite gray</p> 
Product description	<p>PLUS wall tracks are used for mounting of support arms, shower seats and sink brackets, and allows sideways adjustment of these products. See below a selection of products for fitting to the wall track.</p> <p>PLUS wall track, 94.5", for horizontal mounting. The cover plate hides bolts, wires and simplifies cleaning. Materials: Aluminium and polystyrene. Excl. mounting kit.</p>
Adjustability	Fixed mounting
Product capacity	Maximum load: Up to 2200 lbs. depending on the wall type.
Materials	The wall track is primarily made of 10-micrometer anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer.
Ergonomy	The track distributes the weight of the individual bathroom product, for example a sink bracket, along its entire length. The products can be adjusted and locked on to the track with intuitive locking functions.
Features	The horizontal track makes it possible to shift the product along the wall to generate extra space as needed. The front cover plate conceals mounting bolts and any cables.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



PRESSALIT®

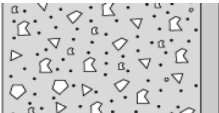
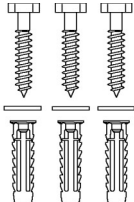

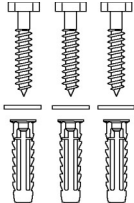
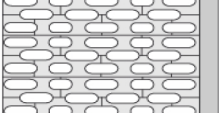
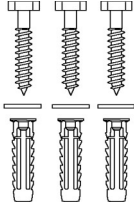
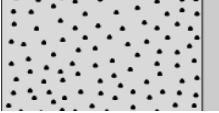
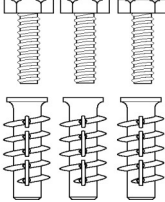
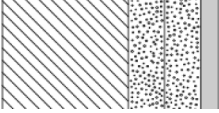
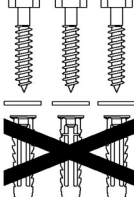
Product	PLUS wall track, 94.5"
Item number	R9824112
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: CaviCide</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
Declaration of conformity	<p>This product fulfils all relevant requirements in European Regulation (EU) 2017/745 on medical devices in risk class I. This product is considered to be an accessory for a medical device, as it is not itself a medical device, but is intended to be used together with several particular medical devices to specifically enable the medical devices to be used in accordance with their intended purposes.</p>



PRESSALIT®

Product	PLUS wall track, 94.5"
Item number	R9824112

Mounting in different wall types

	<p>Concrete >B25</p> <p>V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.</p>	
	<p>Solid Brick >= Mz12</p> <p>V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.</p>	
	<p>Solid Sand Lime Brick >= Hlz12</p> <p>V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.</p>	
	<p>Aerated Concrete P4</p> <p>V8670 - Mounting kit, 3 bolts M8 x 35, plugs FTP M 8. To be ordered separately.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</p> <p>V8620 - Mounting kit, 6 lag bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.</p>	



PRESSALIT®

Product	PLUS wall track, 94.5"
Item number	R9824112

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®