

PRESSALIT.

Product	PLUS wall track, grooved, 15.7". For right hand operated fold down grab bars	
Item number	R9864112	
	VVS:615842104	
	NRF:7501021	
	EAN:5708590305974	
	HMI: 71752	
Color	Anthracite gray	
Product description	PLUS wall track, grooved, for vertical mounting, 15.7". For height adjustable, right-hand operated fold down grab bars. Below a selection of products for fitting to the wall track. The cover plate hides bolts and wires and facilitates cleaning. Excl. mounting kit. See Technical datasheet (Downloads) for information on mounting kits suitable for various wall types.	
Adjustability	Fixed mounting	
Materials	The wall track is primarily made of 10-micrometer anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer	
Ergonomy	The vertical track enables height adjustment of the different products. If combined with the horizontal track, the available space in the bathroom can be utilized optimally. In environments with multiple users, the vertical track makes it possible to adjust the different products to individual user heights.	
Features	The front cover plate conceals mounting bolts and any cables.	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	



PRESSALIT.

Product	PLUS wall track, grooved, 15.7". For right hand operated fold down grab bars
Item number	R9864112
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.
	DISINFECTANTS: CaviCide
	The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.



PRESSALIT®

Product	
Item number	

PLUS wall track, grooved, 15.7". For right hand operated fold down grab bars

Dais

R9864112

Mounting in different wall types

moronic wan types	
Concrete >B25 V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
Solid Brick >= Mz12 V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
Solid Sand Lime Brick >= Hlz12 V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
Aerated Concrete P4 V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



PRESSALIT®

Product Item number	PLUS wall track, grooved, 15.7". For right hand operated fold down grabars R9864112	ab
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com