



# PRESSALIT®

|   |  |
|---|--|
| <b>Product</b><br><b>Item number</b>  | PLUS fold down grab bar, counter-balanced, vertical and horizontal adjustable<br>RF585112000   |
| <b>Color</b>  | VVS:615857700<br>NRF:6190755<br>RSK:8980514<br>EAN:5708590309095<br>HMI: 71766<br>NBS:N13:436<br>FDA:ILS/D135866<br>White  |
| <b>Product description</b>  | <p>PLUS fold down grab bar, counter-balanced, 36.2", ergonomically shaped to provide a clear sense of where the fold down grab bar ends. Right hand operated. Height adjustable by 9.5" and adjustable horizontally. Detachable and foldable. Anthracite cover plate for vertical wall track.</p> <p>Materials: the grab bars are constructed of 10-micrometer anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core</p> <p>For mounting on horizontal wall track (not included).</p> <p>PLUS fold down grab bar, counter-balanced, 36.2", ergonomically shaped to provide a clear sense of where the fold down grab bar ends. Right hand operated. Height adjustable by 9.5" and adjustable horizontally. Detachable and foldable. Anthracite cover plate for vertical wall track.</p> <p>Materials: The arms are constructed of 10-micrometer anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core</p> <p>For mounting on horizontal wall track (not included).</p> |
| <b>Adjustability</b><br><b>Adjustment height</b><br><b>Product capacity</b> | <p>Vertical and horizontal adjustable</p> <p>9,46 inch</p> <p>Maximum load: 330 lbs.<br/>When the fold down grab bar is loaded with 330 lbs, each wall fastener in the horizontal wall track is loaded with 97 lbs.</p> <p><i>PLUS fold down grab bars have been tested statically with 1540 lbs (close to the wall), 935 lbs (in the middle) and 495 lbs (the outermost point).</i></p>   |



# PRESSALIT®

|                    |  |
|--------------------|--|
| <b>Product</b>     | PLUS fold down grab bar, counter-balanced, vertical and horizontal adjustable  |
| <b>Item number</b> | RF585112000  |
| <b>Materials</b>   | <p>The grab bars are primarily constructed in 10-micrometer anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core.</p> <p>The hinge arm is also made of soft SEBS.</p> <p>Internal components and fasteners are primarily made of stainless steel.</p> <p>The hinge barrel is capped with ABS polymer.</p> |
| <b>Ergonomy</b>    | <p>The oval shape gives an easy and comfortable surface to lean and push against.</p> <p>The end of the support arm is enlarged so that it is easier to pull forward from a sitting position and also provides a clear sense of where the support arm ends.</p> <p>Soft and slip-resistant material that has a warm feel.</p>  |
| <b>Features</b>    | <p>Folds against the wall.</p> <p>The underside is shaped for the mounting of various accessories.</p>   |
| <b>Warranty</b>    | <p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>   |



# PRESSALIT®

|                    |  |
|--------------------|--|
| <b>Product</b>     | PLUS fold down grab bar, counter-balanced, vertical and horizontal adjustable  |
| <b>Item number</b> | RF585112000  |
| <b>Cleaning</b>    | <p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b><br/>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningssprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%</p> <p><b>NON-DISINFECTANTS:</b><br/>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b><br/>Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p> |



# PRESSALIT®

|                    |   |
|--------------------|---|
| <b>Product</b>     | PLUS fold down grab bar, counter-balanced, vertical and horizontal adjustable   |
| <b>Item number</b> | RF585112000   |
|                    | <p><b>Intended use</b><br/>The fold down grab bar is used as a support for sitting down or getting up from a toilet, or for sideways transfer from a wheelchair.</p> <p><b>Operating environment</b><br/>For indoor use in a wet environment, such as a bathroom or a washroom, in private homes, nursing homes and hospitals.<br/>For institutional use, the expected life time is 10 years.<br/><i>Life cycle test performed according to ISO 17966:2016.</i></p> <p>For maintenance, please refer to operation and maintenance manual, which is enclosed with the product, and which can also be found on the website under "Downloads".</p> |

[pressalit.com](http://pressalit.com)

# PRESSALIT®