


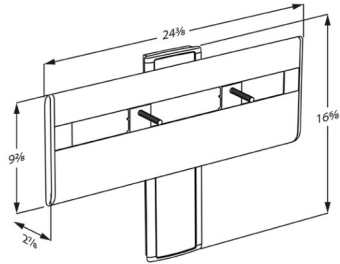

PRESSALIT®

Product Item number	PLUS sink bracket with lever control, manually height adjustable R4550182
Color Product description	<div>  </div> <p>Pale gray</p> <p>PLUS sink brackets are available in electrically, pneumatically and manually height adjustable versions. They are suitable for wall-mounting or for fitting to Pressalit PLUS wall tracks.</p> <p>PLUS sink bracket. Manually height adjustable by 9.5" via lever control. Recommendation: Flexible feed and waste system R2085/R2087. Pale gray cover plate for vertical wall track. Materials: Aluminium and ABS polymer. Incl. mounting bolts for sink, excl. mounting kit for wall.</p>
Adjustability Adjustment height Product capacity	<p>Height adjustable</p> <p>9,46 inch</p> <p>This product has been tested statically up to 440 lbs. As the European standard for technical aids for disabled, EN 12182:2012, requires a static test with safety factor 1.5, the nominal maximum load for this product is therefore 300 lbs.</p>
Materials Ergonomy Features	<p>The sink bracket is primarily made of 3/8" anodized aluminium. The ends of the bracket cover are capped with ABS polymer.</p> <p>The sink bracket is compatible with all types of bolt-mounted sink. It's rounded, sleek design makes it easy to clean thoroughly.</p> <p>If the height of the sink only needs to be adjusted occasionally, a manually adjustable sink bracket could be the best solution. Adjustment involves supporting the weight of the sink and part of the bracket. If frequent height adjustment is anticipated, use pneumatic-assisted sink bracket R4650 or electrically operated sink bracket R475x, instead.</p>
Warranty	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>



PRESSALIT®

Product	PLUS sink bracket with lever control, manually height adjustable
Item number	R4550182
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>



PRESSALIT®

Product	PLUS sink bracket with lever control, manually height adjustable
Item number	R4550182

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®