

## PRESSALIT.

Product	VALUE II shower seat, fixed height	
Item number	R1610000112	
	VVS:777525012	
	RSK:8162781	
	EAN:5708590355108	8%
	HMI: 124483	
Color	Anthracite gray	17%
Product description	VALUE II folding shower seat and backrest create a simple solution for users who need to sit down while showering.	
	VALUE II shower seat. Fixed height. The seat is foldable. Materials: Diecast aluminium, Rilsan-coated stainless steel and vacuum-moulded polyethylene (PE). Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.	
Adjustability	Fixed mounting	
Product capacity	Maximum user mass: 275 lbs.	
	When the seat is loaded with 275 lbs, each wall fastener is loaded with 209 lbs.	
The seat has been tested according to the applied as		y with safety factor 1.5, which is a requirement products standards.
Materials	The wall profile is made of diecast aluminium, and the frame is made of Rilsan-coate stainless steel. Rilsan provides good protection against corrosion and is comfortable to touch and easy to clean.The seat is made of vacuum-moulded polyethylene (PE).	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	



## PRESSALIT.

Product	VALUE II shower seat, fixed height R1610000112	
Item number		
Cleaning		
	This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being	
	damaged. For surface disinfection, chlorine-based disinfectants may be used according to th product specification and safety precautions for these products.	
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.	
	DISINFECTANTS:	
	Clinimax Difficil-S	
	Erisan OXY+	
	Sprit (husholdningssprit) Suma Bac D10	
	Virkon	
	Rodalon Desinfektionsspray	
	Lysoformin Spezial	
	CaviCide	
	Sufanios Premium	
	Tvättsprit 70% WipeClean UniversalCleaning	
	Chrolorine Disinfection Wipes	
	Klorilli (Kiilto Clean)	
	NON-DISINFECTANTS:	
	Suma Star free D1	
	Taski 100	
	Taski 200	
	Milizid Citro	
	The product must be wiped down with a soft rag wrung out in clear water immedia after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.	

## pressalit.com

