

PRESSALIT.®

Product	PLUS shower seat 450 with aperture, manually height adjustable	
Item number	R7425112	
	VVS:777515612	
	NRF:6317121	
	EAN:5708590359748	
	HMI: 124344	
	FDA: INN/D135865	
Color	Anthracite gray	
Product description	PLUS shower seats are offered in a number of colours. They are available as both fixed height, electrically adjustable and manually adjustable, with and without aperture, and with an option of back and armrests. They are suitable for wall-mounting or for fitting to Pressalit PLUS wall tracks.	
	PLUS shower seat 450 with aperture, with backrest. Manually height adjustable by 15", without the use of tools. The seat is foldable. Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminic profile. Excl. mounting kit. Replaces R7421	
Adjustability	Height adjustable	
Adjustment height	14,97 inch	
Product capacity	Maximum user mass: 440 lbs. When the seat is loaded with 440 lbs, each wall fastener is loaded with 40 lbs.	
	The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.	
Materials	The shower seat is made of polyurethane (PU) foam with a plywood core and this is covered with a moulded coated surface that is protected using a clear PU lacquer. The backrest is made of polyurethane (PU) foam protected using a clear PU lacque Internal components and fasteners are primarily made of stainless steel. The hinge barrel is capped with ABS.	
Ergonomy	Ergonomically designed seat - contact surface are 17.7" x 17.2". Soft and slip-resistant material that has a warm feel.	



PRESSALIT.

Product Item number	PLUS shower seat 450 with aperture, manually height adjustable R7425112	
Features	The seat can fold up when not in use or to maximize floor space. The seat and backrest can be adjusted simultaneously thanks to a safe and easy click system.	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.	
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.	
	DISINFECTANTS: CaviCide	
	The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.	
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.	
Declaration of conformity	This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.	



PRESSALIT.

Product				
Item number				

PLUS shower seat 450 with aperture, manually height adjustable


R7425112


Mounting in different wall types

2 7 2 2 2 4	
0.0000	

Concrete >B25


V8612 - Mounting kit, 10 lag bolts $\emptyset 8$ x 70, plugs SX10. To be ordered separately.

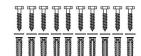


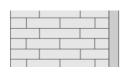

Solid Brick >= Mz12


V8612 - Mounting kit, 10 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.

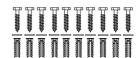

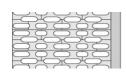

Solid Sand Lime Brick >= Hlz12

V8612 - Mounting kit, 10 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.

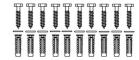




Aerated Concrete P4

V8662 - Mounting kit, 10 countersunk bolts M8 x 35, plugs FTP M 8.

To be ordered separately.

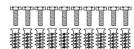




Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8612 - Mounting kit, 10 lag bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed.

To be ordered separately.

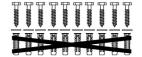



PRESSALIT®

Product Item number	PLUS shower seat 450 with aperture, manually heigl	nt adjustable
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	

pressalit.com