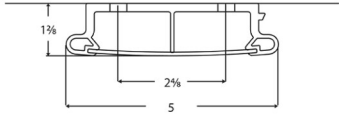




<b>Product</b> <b>Item number</b>	PLUS wall track, 23.6", right-grooved, for vertical mounting R9876112
<b>Color</b>	VVS:615843106 NRF:7501074 EAN:5708590365978 HMI: 124192  Anthracite gray
<b>Product description</b>	  Below a selection of products for fitting to the wall track.  PLUS wall track, 23.6", right-grooved, for vertical mounting. For height adjustable shower seats and folding seats. The cover plate hides bolts and wires and facilitates cleaning. Materials: Aluminium and polystyrene. Excl. fixing kit.
<b>Adjustability</b>	Fixed mounting
<b>Materials</b> <b>Ergonomy</b> <b>Features</b>	The wall track is primarily made of 10-micrometre anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer.  The vertical track enables height adjustment of the different products. If combined with the horizontal track, the available space in the bathroom can be utilized optimally. In environments with multiple users, the vertical track makes it possible to adjust the different products to individual user heights.  The front cover plate conceals mounting bolts and any cables.
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



# PRESSALIT®

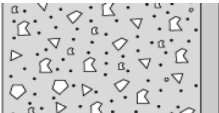
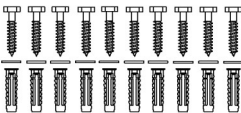

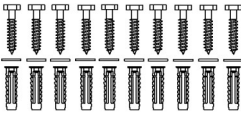
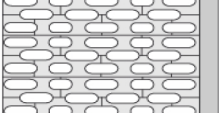
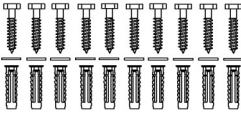
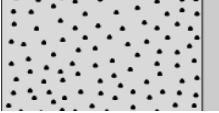
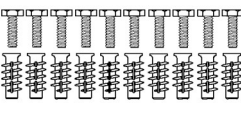
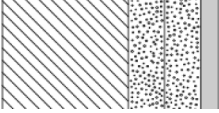
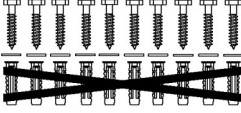
<b>Product</b> <b>Item number</b>	PLUS wall track, 23.6", right-grooved, for vertical mounting R9876112
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> CaviCide</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
<b>Declaration of conformity</b>	<p>This product fulfils all relevant requirements in European Regulation (EU) 2017/745 on medical devices in risk class I. This product is considered to be an <b>accessory</b> for a medical device, as it is not itself a medical device, but is intended to be used together with several particular medical devices to specifically enable the medical devices to be used in accordance with their intended purposes.</p>



PRESSALIT®

<b>Product</b>	PLUS wall track, 23.6", right-grooved, for vertical mounting
<b>Item number</b>	R9876112

## Mounting in different wall types

	<b>Concrete &gt;B25</b> V8612 - Mounting kit, 10 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Solid Brick &gt;= Mz12</b> V8612 - Mounting kit, 10 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Solid Sand Lime Brick &gt;= Hlz12</b> V8612 - Mounting kit, 10 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Aerated Concrete P4</b> V8662 - Mounting kit, 10 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b> V8612 - Mounting kit, 10 lag bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



# PRESSALIT®

<b>Product</b> <b>Item number</b>	PLUS wall track, 23.6", right-grooved, for vertical mounting R9876112
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b></p> <p>Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b></p> <p>Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>
	<p><b>Drywall (2 x 12,5 mm) water-resistant</b></p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>

pressalit.com

Pressalit A/S  
 Pressalitvej 1  
 DK-8680 Ry  
 Tel.: +45 8788 8788  
 Fax: +45 8788 8789  
 E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®